




EARMUFFZ
HEARING FOR LIFE

User Guide

How to use your Earmuffz Case

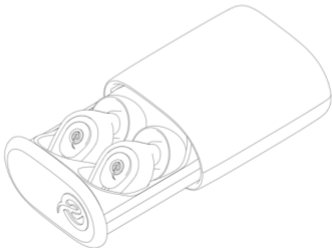

Opening your Earmuffz Case



Follow these steps to open your case correctly:



1

Hold the case horizontally. Make sure the logo is facing you and the case is upright.



2

Press to open. Apply gentle pressure to the back of the case – the spring mechanism will slide the front insert open, presenting both earplugs.

Tip: Press firmly but gently – the case will slide open smoothly when held the right way, with the logo facing you.

Opening your Earmuffz Case



Follow these steps to remove your earplugs correctly:



1

Hold the case securely in your hand.




2

Push on the back of the lid to pop it open.



3

Gently lift one earplug out from the silicone insert.

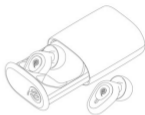

4

Your earplug is now out and ready to insert in your ear.

Tip: Make sure the case is upright, with the logo on the front facing you, for easy opening and earplug removal.

Cleaning your Earmuffz Case



Clean them properly by following these tips:



Use a damp cloth to gently clean the case.



Allow all parts to dry completely before reattaching the ear tips.



Do not soak the earplugs in water and avoid using harsh chemicals or sharp tools.

Disclaimer



The information provided in this user guide is for general guidance and informational purposes only. While we strive to ensure the accuracy and completeness of the content, Earmuffz makes no warranties or representations, express or implied, about the suitability, reliability, or accuracy of the information contained herein.

Use of our products should always follow the safety instructions and guidelines provided. Improper use may result in reduced effectiveness or personal injury. Earmuffz is not liable for any damage, loss, or injury arising from the misuse of our products or failure to follow instructions.

Product specifications, designs, and user instructions are subject to change without prior notice. For the latest version of this manual or additional support, please visit our website.



See information supplied
by the manufacturer



2957



EARMUFFZ

HEARING FOR LIFE