




EARMUFFZ
HEARING FOR LIFE

User Guide


How to use your Earmuffz Neck Strap

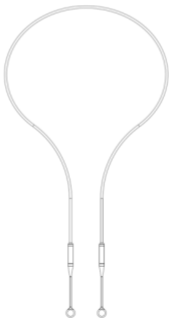

Attaching Your Earmuffz Earplugs



Follow these steps to attach your earplugs to the neck strap properly:

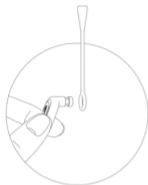

1

Remove the silicone tip
and hold the earplug
securely.



2

Slide the earplug into
the silicone end of the
neck strap.



3

Reattach the silicone
ear tip and you're ready
to go.

Tip: Ensure the earplug is aligned correctly so it passes smoothly through the silicone neck-strap end without twisting.

How to Ensure Proper Assembly



Follow these steps to ensure your neck strap and earplugs are assembled properly:



1

Check the earplug orientation.

Ensure the earplug sits as shown in the illustration, with the neck-strap aligned correctly.



2

Make sure the silicone tip is centered.


The silicone should sit evenly on the earplug stem for a secure fit.

Magnetic Safety Clasp

Follow these steps to detach your neck strap correctly:

1

How the detachable tips work.
The silicone tips attach securely to the neck strap using a push-fit connection. To remove a tip, gently pull the silicone section away from the connector. To reattach, press the tip firmly back into place until secure.

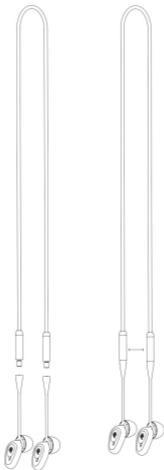


2

Safety clasp.
The clasp is magnetic both laterally and horizontally for a secure but safe connection. Pull the strap ends apart to release.

Tip: Use light, even pressure. The clasp is designed to open smoothly.

Understanding your Earmuffz Neck Strap

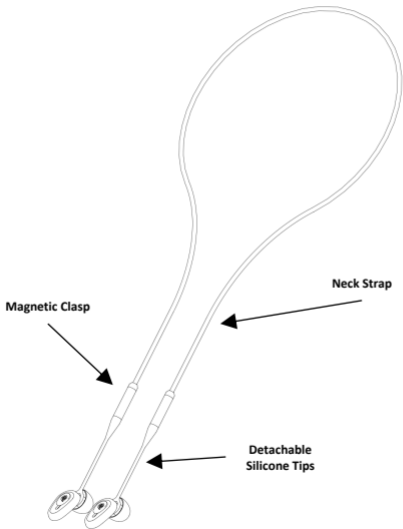

Cleaning your Neck Strap



Clean your Neck Strap properly by following these tips:



Use a damp cloth to gently clean the neck strap.
Repeat the process for the silicone tips.



Allow all parts to dry completely before reattaching the earplugs.



Do not soak the neck strap in water and avoid using harsh chemicals or sharp tools

Disclaimer



The information provided in this user guide is for general guidance and informational purposes only. While we strive to ensure the accuracy and completeness of the content, Earmuffz makes no warranties or representations, express or implied, about the suitability, reliability, or accuracy of the information contained herein.

Use of our products should always follow the safety instructions and guidelines provided. Improper use may result in reduced effectiveness or personal injury. Earmuffz is not liable for any damage, loss, or injury arising from the misuse of our products or failure to follow instructions.

Product specifications, designs, and user instructions are subject to change without prior notice. For the latest version of this manual or additional support, please visit our website.



See information supplied
by the manufacturer



2957



EARMUFFZ

HEARING FOR LIFE